User information

Nitrile examination gloves – CAT.3

Article G921

Available sizes: XS (5/6); S (6/7); M (7/8); L (8/9); XL (9/10)

This product is classed as Personal Protective Equipment (PPE) according to European PPE Regulation 2016/425.

Notified Body responsible for certification is SATRA Technology Europe, Bracetown Business Park Clonee, Dublin D15YN2P, Ireland NB 2777.

The Notified Body is responsible for both EU Type Examination and ongoing conformity.

The protection is claimed for the EU market.

PERFORMANCE AND LIMITATION OF USE – This product has been tested in accordance with EN ISO 374-1: 2016+A1:2018, EN ISO 374-5:2016 and EN 420:2003+A1:2009+A1: 2009 and has achieved the following performance levels:

EN ISO 374-1: 2016 (Type B)

	Permeation Level	EN374-4: 2013 Degradation %
Sodium hydroxide 40% (K)	6	4.0
n- Heptane (J)	1	47.3
37% Formaldehyde (T)	4	30.1
70% Isopropanol	2	61.8
30% Hydrogen Peroxide (P)	4	18.7

Minimum breakthrough time min	Permeation performance levels
>10	1
>30	2
>60	3
>120	4
>240	5
>480	6

EN374-4: 2013 Degradation results indicate the change in puncture resistance of the gloves after exposure to the challenge chemical

EN ISO 374-5: 2016

Protection against bacteria and fungi
Protection against viruses

Level

Pass
Pass

This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals.

The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. It can be different if the chemical is used in a mixture.

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves.

The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen.

Before usage, inspect the gloves for any defect or imperfections”.

For single use only.

The composition of the glove has no substances known to cause an allergy reaction.


Category III product.

FITTING AND SIZING – These gloves are designed to protect the hand surface from chemical splashes. Therefore, the length of the gloves is below EN requirements of total minimum glove length, and deems as ‘Fit for Special Purpose. Please, choose the appropriate glove size to avoid hand fatigue, see the hand guide on the wrapper, turn the cuff down to avoid drainage of chemicals onto the arm.

INSTRUCTIONS FOR MAINTENANCE AND EXPIRATION DATE: The glove needs to be stored in its original packaging, in a dry place and away from heat source. Open boxes should be kept away from fluorescent and sun light

CLEANING: no cleaning possible

DECLARATION OF CONFORMITY CAN BE FOUND AT: WWW.TIGRE.IT

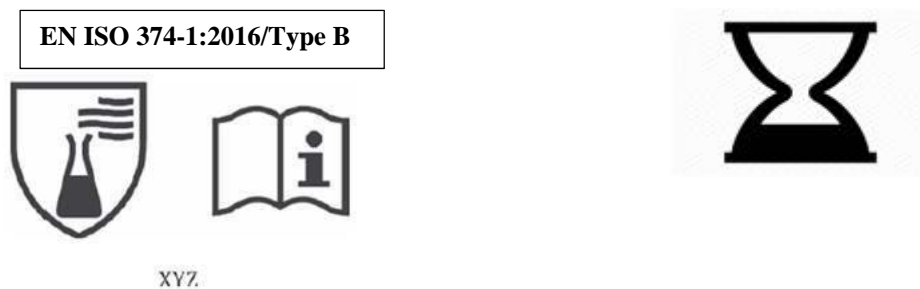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